

*Rehab*Boost



QUICK STEP-BY-STEP INSTRUCTIONS



THERAPIST

Download on the
App Store


ANDROID APP ON
Google play





In this document you'll see quick step-by-step instructions on how to use the **Rehab Boost App** and **Rehab Boost Professional Dashboard**.

You will be guided through a comprehensive simplified instruction on how to use this product type within our AI Platform.

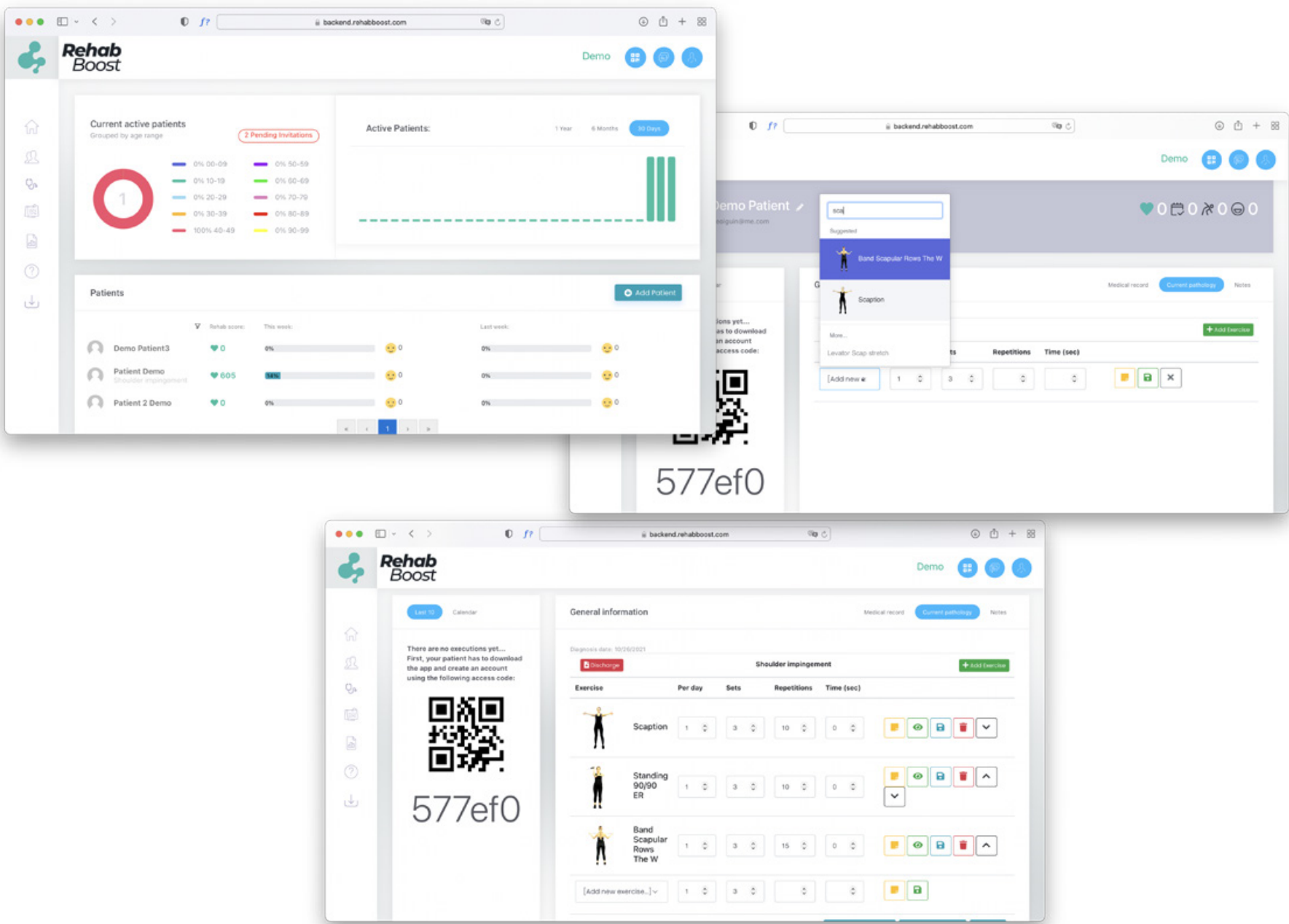


You can download the **Rehab Boost App** from the corresponding appstores by scanning this QR code.





01

From any web browser on a computer or tablet, login to:
backend.rehabboost.com

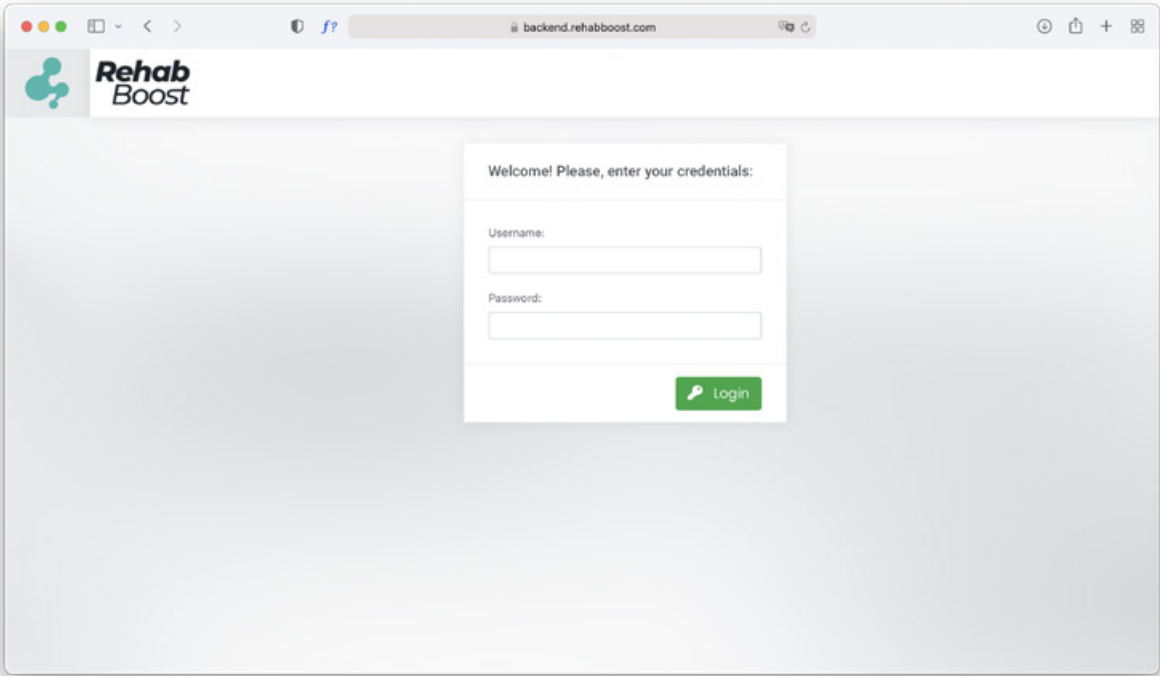


02

Your **dashboard** will include all active patients, their Rehab Scores and all compliance metrics.

To ADD A NEW PATIENT, simply click on the **“Add Patient”** Button.


+ Add Patient

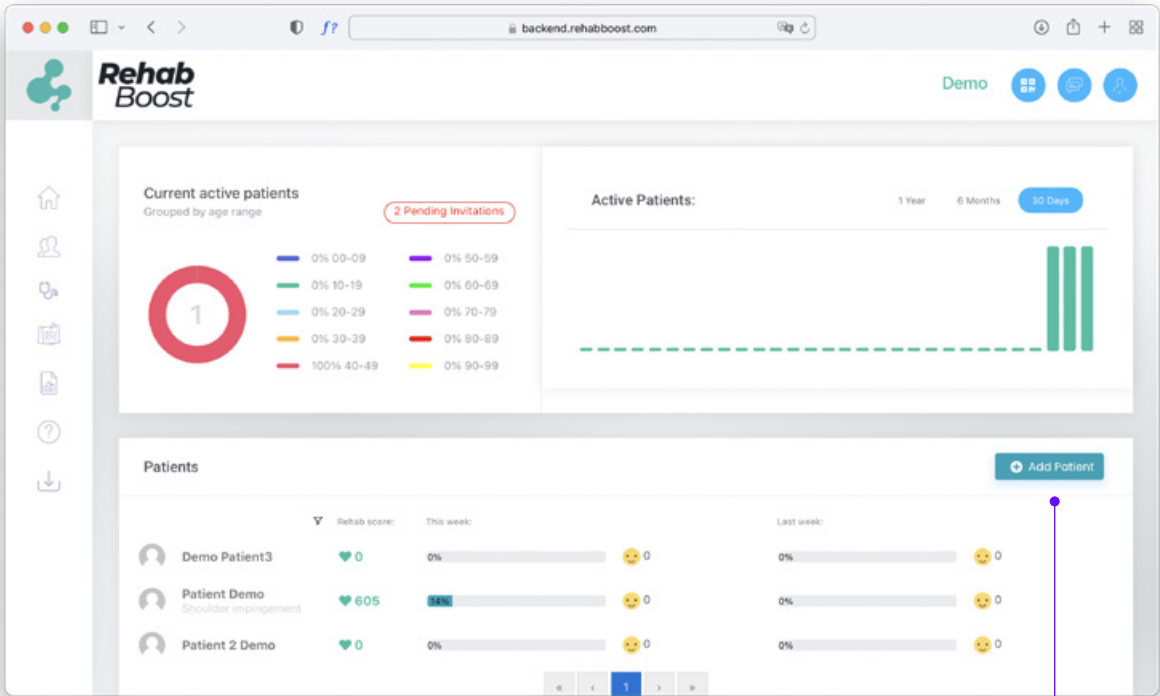


03

Enter required patient information and press **“Create Patient and Send Invitation Code”** button

✓ Create Patient and Send Invitation Code

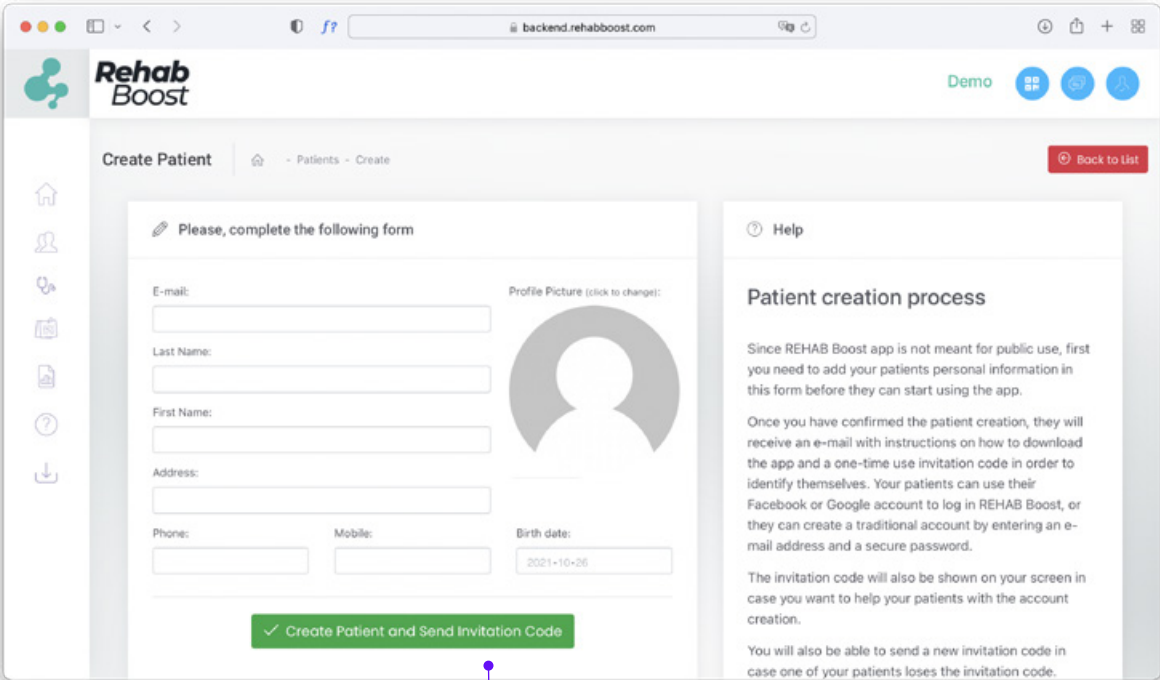


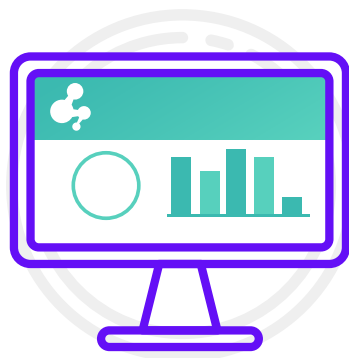

I AM A
THERAPIST



04

You will have created a new patient and generated an "Invitation Code"

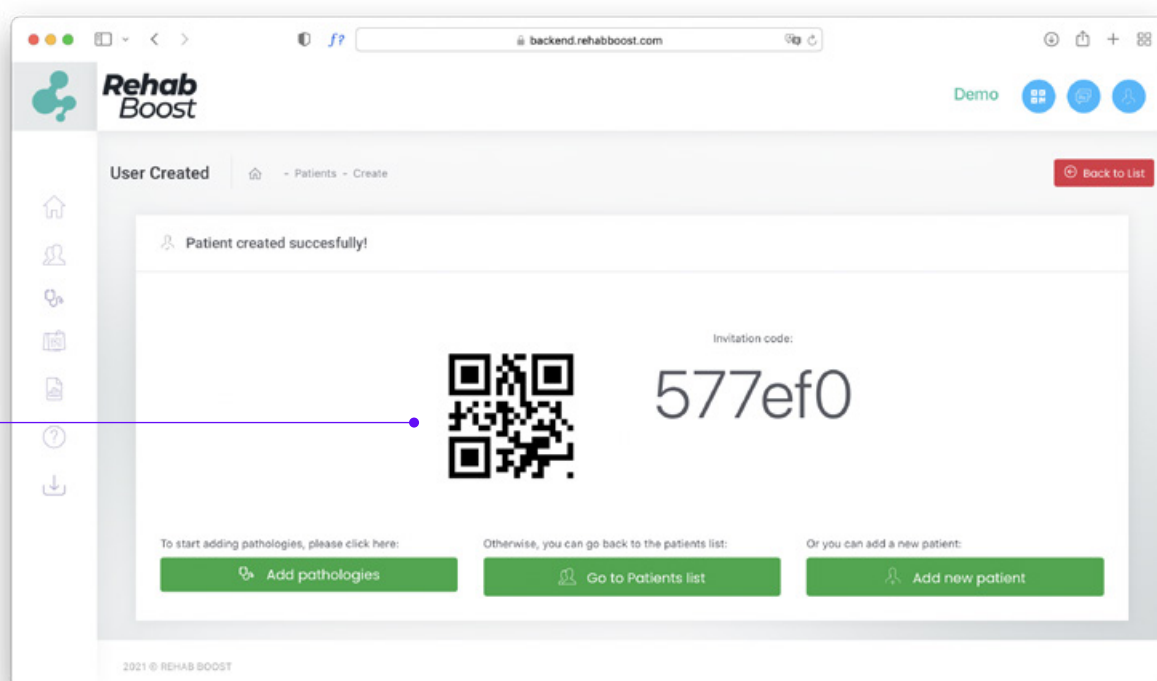

05


Have patient **download the Rehab Boost mobile App** from the Apple AppStore or Android Store, patient should **create an account** and point app at the generated "Invitation Code" you just created. Patient can enter the code manually or scan the provided QR code with phone's camera.

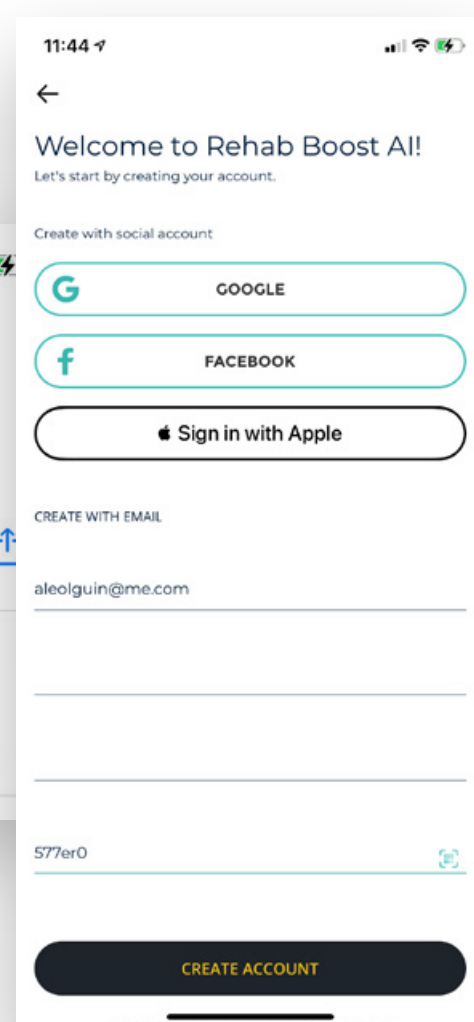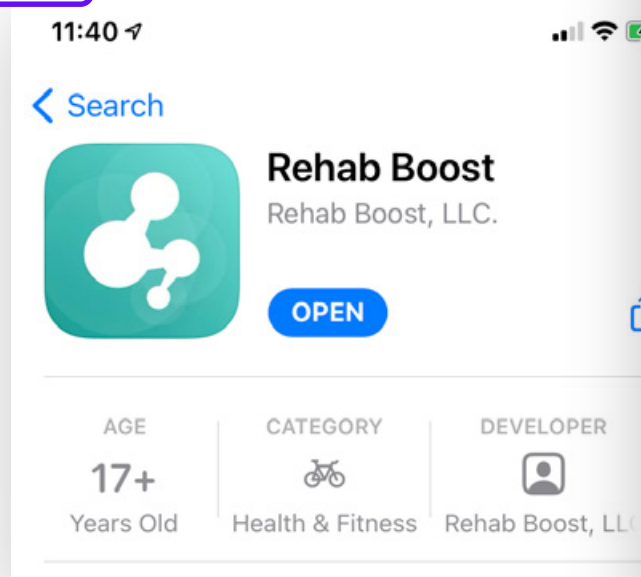

06

From the Therapist Dashboard, click **"Add Pathologies"** button.

From the patient dashboard, click **"Add Pathologies"** button.

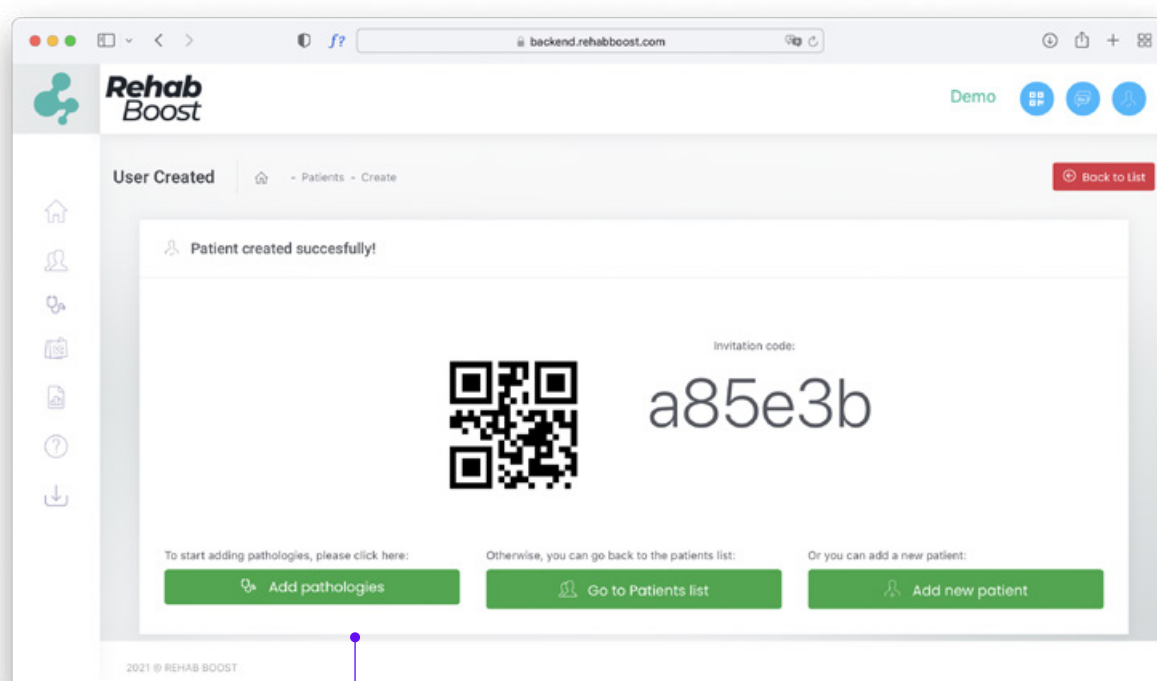


Add pathologies




07

A drop-down menu will be provided to select desired pathology and treatment plan. **Choose Pathology** [For example: Shoulder impingement] and press the **“Add”** button

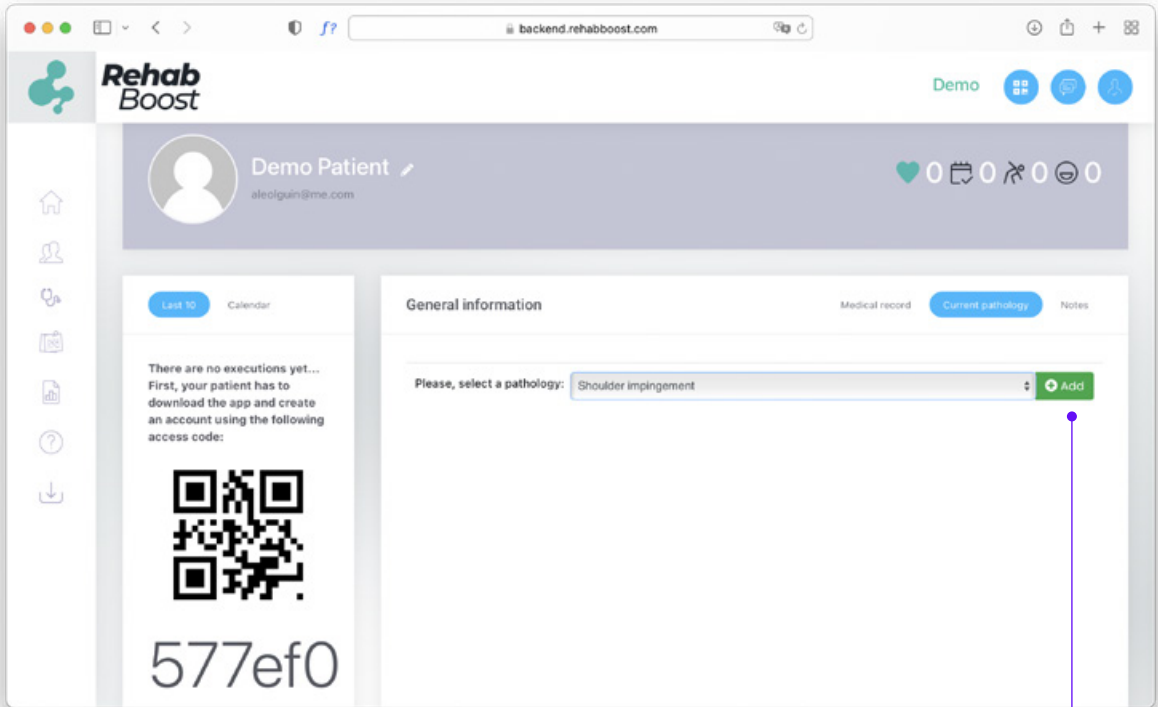


08

Choose **Exercise** [For example: Scaption] and press the **SAVE** button. Repeat to add as many exercises as you wish and press **SAVE** after each selection.

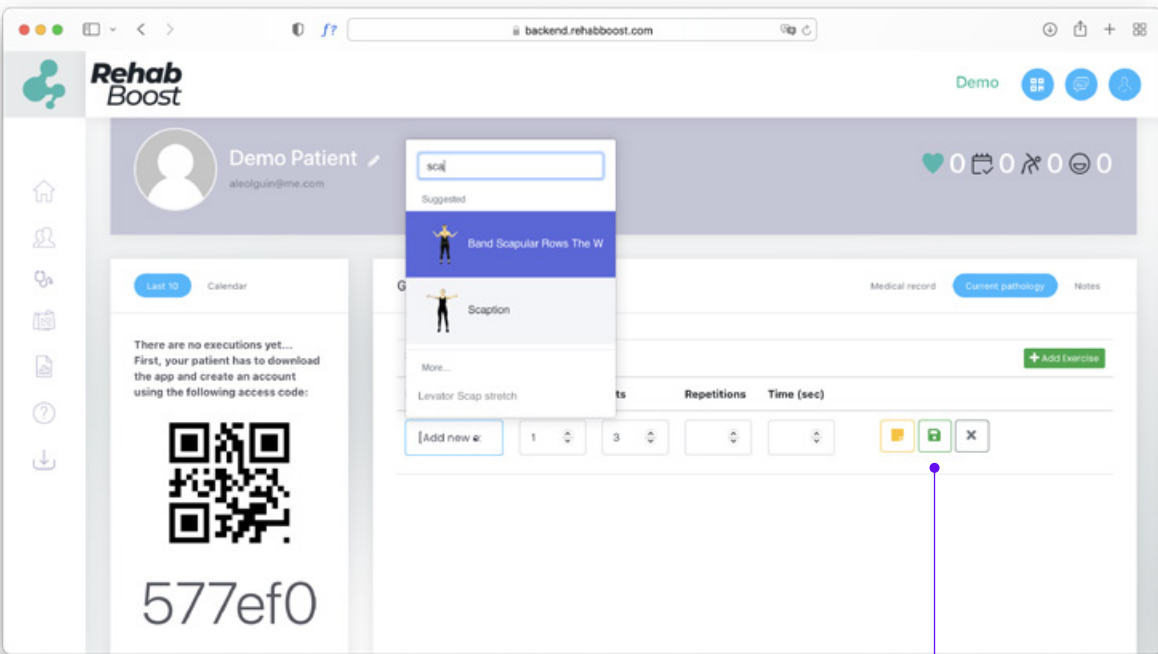

09

Repeat to add as many exercises as you wish and press **SAVE** after each selection

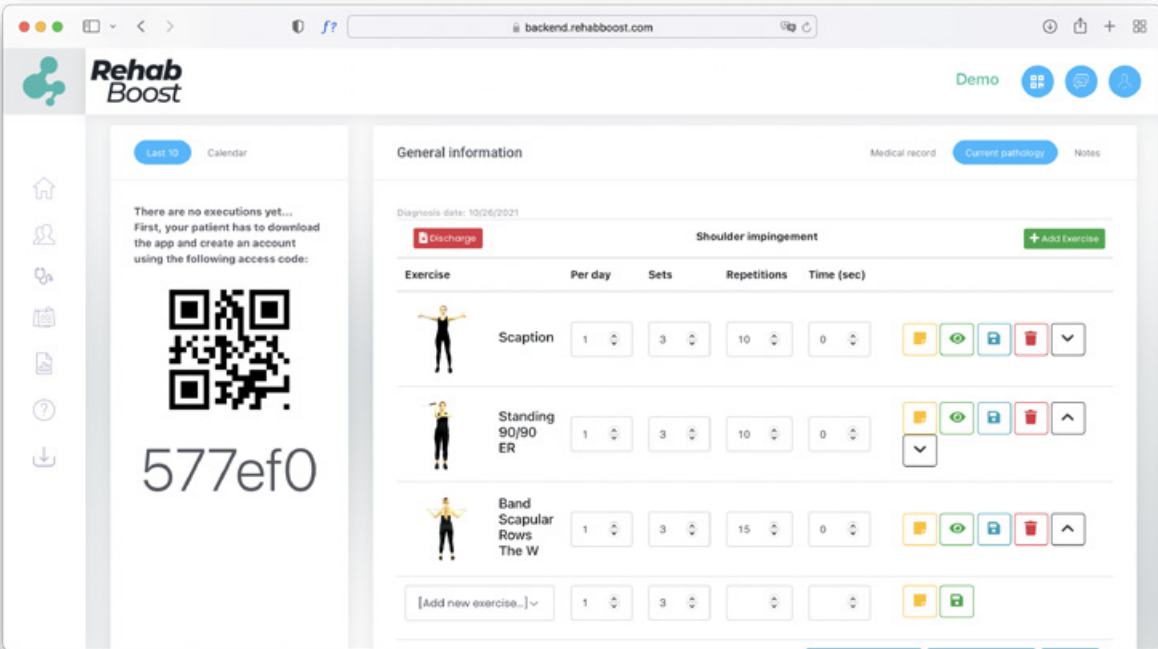



Your treatment plan will be **automatically** populated in the Patient App so can begin **Artificial Intelligence Rehabbing** anytime, anywhere.

